



Instructions for Use

Adhesive Remover (Spray)

Iatriki Tomi LP - Sealtec

Add.: Marathonos str.Efkarpia 56429 Thessaloniki Greece
EU

Tel.: +30 2310 930 500

www.sealtec.gr

info@sealtec.gr

Application:

Apply to the flange surface which stick to skin, to decrease the adhesive ability, help to peel off from skin.

Ingredients: Siloxane blends

Precautions:

1. Please strictly follow the product instructions for use, please consult your Stoma nurse or doctor if necessary;
2. Please use it under the guidance of medical staff. Our company is not responsible for all damages and losses caused by not using the product in accordance with the instructions for use;
3. This product should be applied to the skin around the stoma without open wounds;
4. If you have skin allergies, stop using it immediately;
5. Tighten the cap immediately after use to prevent contamination;
6. Avoid direct inhalation.

Storage conditions:

Three years when keep in cool and dry place, avoid direct sunlight, far away from fire.

Disposal method:

According to local regulations, such spray bottles should be discarded with bottles with same classification.


Caution:

This product is a non-sterile product and should not be used in people with damage to the immune system, such as leukemia patients, who have their immune system suppressed by drugs after organ transplantation. Cannot be used in premature babies or infants who receive care in a nursery box.



Use steps:

1. Open the bottle cap, press down the nozzle, and spray the solution evenly to the skin area.

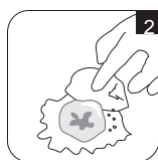

2. let the solution stay on skin for a while, then remove the residual glue together with solution slightly by gauze or towel.

3. After use, cover the cap.

Symbols illustration:

	MANUFACTURER		TEMPERATURE 10C-30C
	MANUFACTURING NUMBER		PROTECT FROM SUN
	NON STERILE		NON REUSABLE
	MANUFACTURE DATE		REFER THE INSTRUCTIONS FOR USE
	EXPIRATION		PROTECT FROM WATER
	CE MARK		MEDICAL DEVICE
	REFERENCE NUMBER		DO NOT USE IF THE PACKAGE IS DAMAGED
	LATEX FREE		WARNING MUST READ INSTRUCTIONS FOR USE
	FAR AWAY FROM FIRE		

Revision date:
2021-05-01/v1



Instructions for Use

Adhesive Remover (Wipe)



Iatriki Tomi LP - Sealtec

Add.: Marathonos str.Efkarpia 56429 Thessaloniki
Greece EU

Tel.: +30 2310 930 500

www.sealtec.gr

info@sealtec.gr

Application:

Apply to the flange surface which stick to skin, to decrease the adhesive ability, help to peel off from skin.

Ingredients: Siloxane blends

Precautions:

1. Please strictly follow the product instructions for use, please consult your Stoma nurse or doctor if necessary;
2. Please use it under the guidance of medical staff. Our company is not responsible for all damages and losses caused by not using the product in accordance with the instructions for use;
3. This product should be applied to the skin around the stoma without open wounds;
4. If you have skin allergies, stop using it immediately;
5. Tighten the cap immediately after use to prevent contamination;
6. Avoid direct inhalation.

Storage conditions:

Three years when keep in cool and dry place, avoid direct sunlight, far away from fire.

Disposal method:


According to local regulations.

Caution:

This product is a non-sterile product and should not be used in people with damage to the immune system, such as leukemia patients, who have their immune system suppressed by drugs after organ transplantation. Cannot be used in premature babies or infants who receive care in a nursery box.

Use steps:

1. Wash hands thoroughly before use.
2. Open the sachet, use the wipe apply the solution around the edge of ostomy adhesive and for a few seconds.
3. Gently peel adhesive away from the skin applying additional solution to ease remove if necessary.
4. The solution dries in seconds and will not affect the adhesion of replacement ostomy appliances.

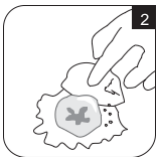

Symbols illustration:

	MANUFACTURER		TEMPERATURE 10C-30C
	MANUFACTURING NUMBER		PROTECT FROM SUN
	NON STERILE		NON REUSABLE
	MANUFACTURE DATE		REFER THE INSTRUCTIONS FOR USE
	EXPIRATION		PROTECT FROM WATER
	CE MARK		MEDICAL DEVICE
	REFERENCE NUMBER		DO NOT USE IF THE PACKAGE IS DAMAGED
	LATEX FREE		WARNING MUST READ INSTRUCTIONS FOR USE
	FAR AWAY FROM FIRE		

Revision date: 2021-05-01/v1