



# Instructions for Use

## Ostomy Bags

 Iatriki Tomi LP - Sealtec

Add.: Marathonos str. Efkarpia 56429 Thessaloniki  
Greece EU

Tel.: +30 2310 930 500

www.sealtec.gr

info@sealtec.gr

### Items:

One-piece pouches for col/ile/urostomy

### Intended Use:

The products are intended to collect output from a stoma.

### Caution:

1. Please strictly follow the instructions for use of this products. If necessary, please consult your ET nurses or doctors;
2. Please use the pouch with convex flange under the guidance of medical professionals;
3. our company accepts no liability for any injury or other losses caused by using the products in a manner which not comply with our company's commendations;
4. It is recommended to change the bag every two days;
5. Don't flush the pouch directly to the toilet, please put them into trash chute.




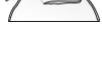





### Storage:

Three years when keep products in cool and dry place. Avoid children's touch.

### Warning:

1. This product is a one-time use product. Reuse or excessive use may have potential risks to users. Reprocessing, cleaning, and sterilizing can all affect the product's characteristics, which may cause other injuries to the user, or enhance the risk of infection;
2. If you have allergies, stop using it and consult with medical professionals;
3. If there is skin defect or damage at the site of adhesion, please use it with caution;
4. Children or unresponsive patients may need the help of others to use the products.


### Use steps:

1.  Clean the skin  
Clean the skin around the stoma with clean water or saline. Dry the skin.
2.  Cutting  
Trace the correct size onto the back of the wafer with the starter hole in the middle. Cut out the hole. Leave only 1/8 to 1/16 gap between hole and stoma to avoid leakage occur.
3.  Seal the drainage  
(Available for ile/urostomy pouches)
  - 3.1  Clamp closure  
Fold the pouch tail by the insert piece of the clamp. Put the insert piece into the groove part to seal the pouch.
  - 3.2  Flex closure  
Roll up the pouch tail by the flex strip about 4-5 times to the out side Direction. Fold and press down the side wings to seal the pouch tightly.
  - 3.3  Velcro closure  
Fold the pouch tail to the outside by the flake, still the one pieces velcro face to each other. Press and make sure the velcro surface seal tightly.
- 3.4  Urostomy pouches  
Insert the plug into the flexible drainage to seal the pouch.
4.  Tear away the glue protect layer  
Tear away the protect layer on the flange.
5.  Wearing the pouch  
Posit the stoma at the center of the flange hole, smooth the skin then stick the flange onto body, Press and smooth the flange from center to around, fix it well to the skin.

6. Empty and clean the pouch (Available for ile/urostomy pouches)

#### 6.1 Ileostomy pouch

Empty the pouch when it's 1/3 to 1/2 full, open the pouch tail, flush the container into toilet. Clean the pouch tail by tap water, dry the pouch outside, seal the pouch as step 3., Continue to use.

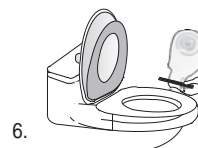

#### 6.2 Urostomy pouch

Empty the pouch when it's 1/3 to 1/2 full, pull the plug out from the drainage, let the container flush into the toilet. Clean the drainage by tap water, seal the pouch as step 3.4.

#### 7. Uninstall the pouch

Press the skin under flange, peel off the flange slightly by another hand. Avoid to stretch the skin. Could using clean water or cleanser to weak the adhesive area and help to peel off the flange.

### Symbols illustration:



# Instructions for Use

## Ostomy Bags

**Iatriki Tomi LP - Sealtec**  
 Add.: Marathonos str.Efkarpia 56429 Thessaloniki  
 Greece EU  
 Tel.: +30 2310 930 500  
 www.sealtec.gr info@sealtec.gr

**Items:**  
**Two-piece pouches for colo/ileo/urostomy**









**Intended Use:**  
 The products are intended to collect output from a stoma.





- Caution:**
1. Please strictly follow the instructions for use of this products. If necessary, please consult your ET nurses or doctors;
  2. Please use the pouch with convex flange under the guidance of medical professionals;
  3. our company accepts no liability for any injury or other losses caused by using the products in a manner which not comply with our company's commendations;
  4. It is recommended to change the bag every two days;
  5. Don't flush the pouch directly to the toilet, please put them into trash chute.

**Storage:**  
 Three years when keep products in cool and dry place. Avoid children's touch.

- Warning:**
1. This product is a one-time use product. Reuse or excessive use may have potential risks to users. Reprocessing, cleaning, and sterilizing can all affect the product's characteristics, which may cause other injuries to the user, or enhance the risk of infection;
  2. If you have allergies, stop using it and consult with medical professionals;
  3. If there is skin defect or damage at the site of adhesion, please use it with caution;
  4. Children or unresponsive patients may need the help of others to use the products.


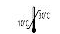





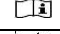








**Use steps:**

1.  Clean the skin around the stoma with clean water or saline. Dry the skin.
2.  Trace the correct size onto the skin barrier with the starter hole in the middle. Cut out the hole. Leave only 1/8 to 1/16 gap between hole and stoma to avoid leakage occur.
3.  Tear away the glue protect layer. Tear away the protect layer on the flange.
4.  Posit the stoma at the center of the flange hole, smooth the skin then stick the flange onto body, press and smooth the flange from center to around, fix it well to the skin.
- 5.1  Clamp closure (Available for ileo/urostomy pouches)  
 5.1 Clamp closure  
 Fold the pouch tail by the insert piece of the clamp. Put the insert piece into the groove part to seal the pouch.
- 5.2  Flex closure  
 Roll up the pouch tail by the flex strip about 4-5 times to the out side direction. Fold and press down the side wings to seal the pouch tightly.
- 5.3  Velcro closure  
 Fold the pouch tail to the outside by the flake, still the two pieces velcro face to each other. Press and make sure the velcro surface seal tightly.
- 5.4  Urostomy pouch  
 Insert the plug into the flexible drainage to seal the pouch.

6.  Put the pouch onto the skin barrier, match the skin barrier with the coupling ring, move and press your finger to the top of the coupling, the pouch will be installed with a click.
7.  Uninstall the pouch  
 Press the skin barrier, peel off the pouch slightly by another hand.
8.  Empty and clean the pouch (Available for ileo/urostomy pouches)  
 8.1 Ileostomy pouch  
 Empty the pouch when it's 1/3 to 1/2 full, open the pouch tail, flush the container into toilet. Clean the pouch tail by tap water, dry the pouch outside, seal the pouch as step 3, Continue to use.  
 8.2 Urostomy pouch  
 Empty the pouch when it's 1/3 to 1/2 full, pull the plug out from the drainage, let the container flush into the toilet. Clean the drainage by tap water, seal the pouch as step 3.4.
9.  Replace skin barrier  
 Press the skin under flange, peel off the flange slightly by another hand. Avoid to stretch the skin. Could using clean water or cleanser to weak the adhesive area and help to peel off.

**Symbols illustration:**

Revision date: 2021-03-01/v3

	MANUFACTURER		TEMPERATURE 10C-30C
	MANUFACTURING NUMBER		PROTECT FROM SUN
	NON STERILE		NON REUSABLE
	MANUFACTURE DATE		REFER THE INSTRUCTIONS FOR USE
	EXPIRATION		PROTECT FROM WATER
	CE MARK		MEDICAL DEVICE
	REFERENCE NUMBER		DO NOT USE IF THE PACKAGE IS DAMAGED
	LATEX FREE		WARNING MUST READ INSTRUCTIONS FOR USE